

ASSEMBLY INSTRUCTIONS

Trapeze bar
Comfort Lift



Bedding Group

These instructions describe the assembly of the trapeze bar holder on the Comfort Lift.

The trapeze bar cannot be combined with a headboard.

With this bracket, the trapeze bar can be mounted on the rear of the Comfort Lift on either the left or right side, and the handle will hang in the middle above the bed.

Bring the lift to its lowest position for safe operation.


Raise the adjustable back section to the highest position so that the top frame on the head side is accessible.

Remove the bolt and bush from the frame on both sides (see photo below).



Slide the bracket as deep as possible into the frame.


Make sure that the recess (see photo below) in the bracket on the left and right is at the bolt position on both sides of the frame.

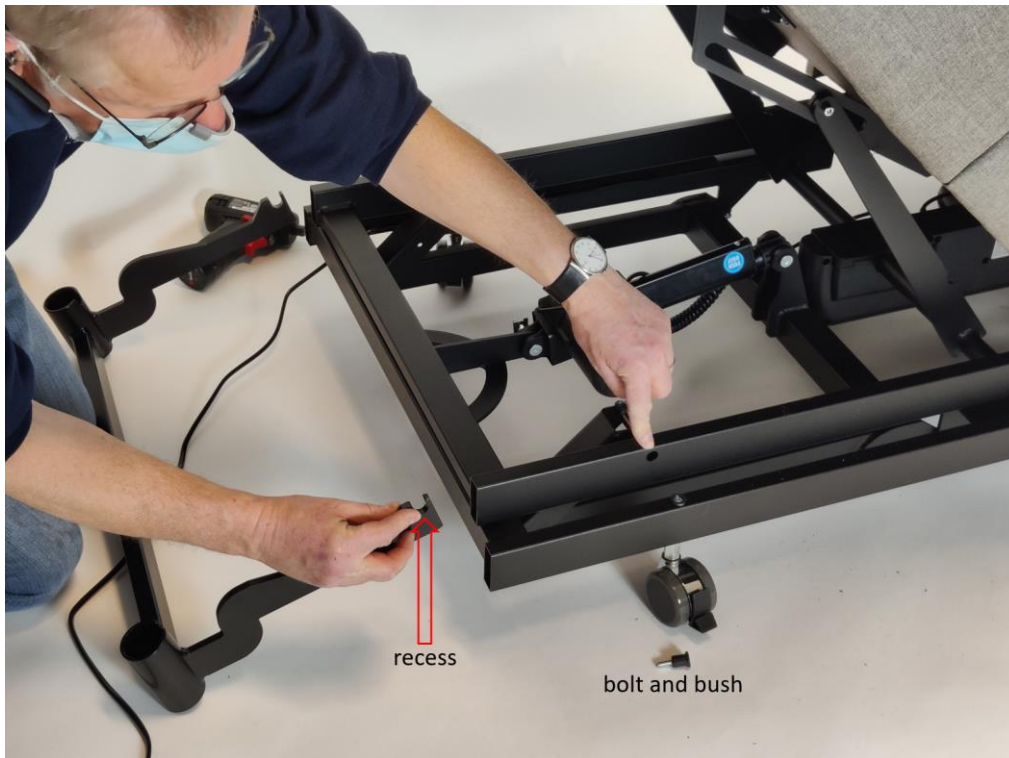

Replace bolt and bush on both sides of the frame.



You can now place the trapeze bar either on the left or the right.

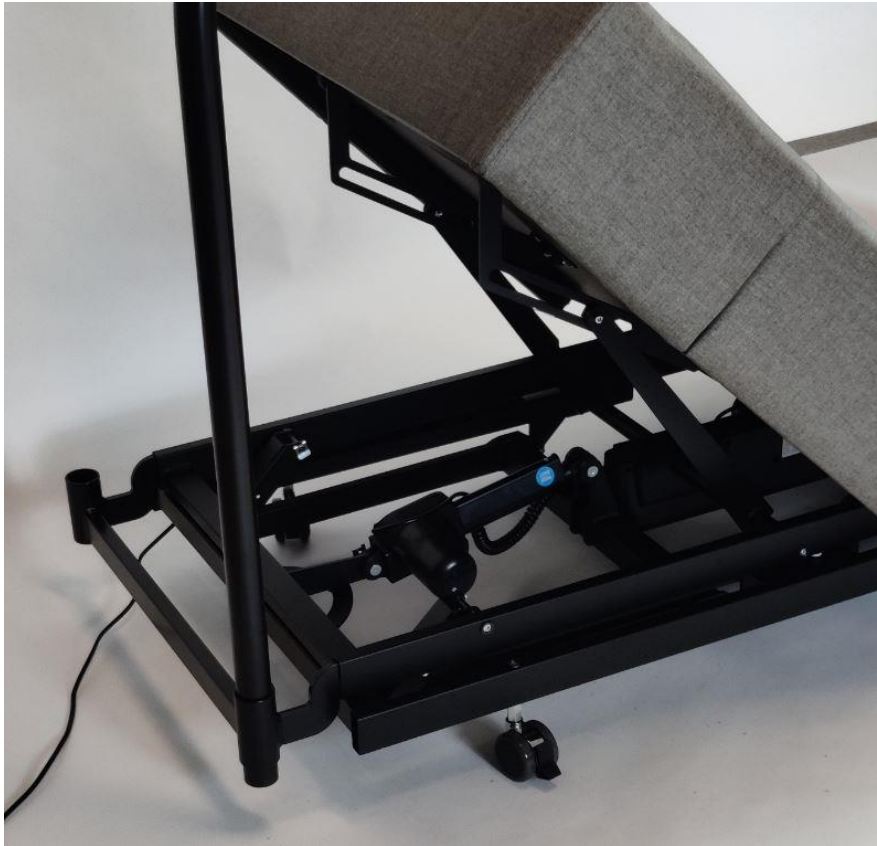

Now, attach the handle to the trapeze bar.



The trapeze bar is in the top frame of the Comfort Lift, so it moves with the box spring.